UNISTRIP™

TECHNOLOGIES, LLC

Control Solution

Product Numbers:

36210 (Low/Level 1)

37320 (High/Level 2)

Intended Use:

UniStrip™ control solution is a red liquid that contains glucose. It reacts with the test strips to produce a test result. You can use either UniStrip™ Control Solution Low (Level 1) or High (Level 2).

Summary:

UniStrip™ control solutions contain certain amounts of glucose that can be tested with your glucose meters. Test results should fall within the control ranges printed on the back of your UniStrip™ test strip vial.

Caution: The control solution ranges are printed on the back of the UniStrip™ test strip vial. They are not target ranges for your blood glucose.

Important Information:

- Use only UniStrip™ control solutions with your meter.
- Check the expiration date on the bottle. Do not use if expired.
- Use within 90 days of the date that you first open it. Record the discard date on the bottle when you first open it to remind you to discard after 90 days.
- For in vitro diagnostic use.

Why Perform a Control Test:

- To ensure that your meter and test strips are working well
- To allow you to practice testing without using your own blood

When to use:

- Once a week (to ensure that your test results remain accurate)
- When you begin using a new vial of test strips
- When test strips have been exposed to extreme environmental conditions
- If you drop the meter
- If you change the battery in the meter

Storage and Handling:

- Control solution should come to room temperature (68-77°F/20-25°C) before testing.
- Store the control solution tightly closed at temperatures below 39-86°F (4-30°C).
- Do not freeze.

Composition:

	Glucose (% w/v)	Nonreactive Ingredients (% w/v)
Low Solution (Level 1)	0.05	99.95
High Solution (Level 2)	0.19	99.81

Test Procedure:

Read your manual before performing a control test.

Follow these steps to perform a control test:

1. Check the expiration and discard dates on your control solution and test strip vials.
2. Shake the control solution bottle well, then remove cap.
3. Squeeze the bottle and discard the first drop then wipe the tip with clean tissue paper or cotton.
4. Squeeze the bottle again to get a second drop onto a clean, non-absorbent surface.
5. Bring the tip of the test strip to touch the drop of solution until the meter beeps.

Caution: To avoid contaminating the control solution with the content of the test strip, do not apply the control solution onto the test strip straight from the bottle.

Results:

After the test, check your control solution bottle to see if it contains a High or Low solution. Compare test results with the range (High or Low) on the back of the test strip vial.

Questionable Results:

If the test result falls outside the range printed on the test strip vial, check the meter owner's manual and repeat the test.

Results that fall outside of the control ranges may indicate:

- An error in how the test was performed
- Control solution bottle not shaken well
- Expired or contaminated control solution
- Control solution that is too warm or too cold
- Test strip deterioration
- Meter malfunction
- Test strips expired

Caution: If you continue to get out-of-range results, it means that the system or the control solution may not be working properly. DO NOT use the system to test your blood glucose level. Customer service: Contact your local supplier or visit www.unistrip-tech.com.

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